
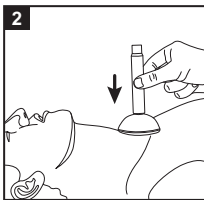
# LARYVOX® SOS MASK

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## INSTRUCTIONS FOR USE

### BILDER / PICTURES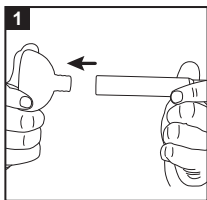

### PIKTOGRAMM-LEGENDE / PICTROGRAM LEGEND

Bestellnummer /  
Catalogue number

Gebrauchsanweisung beachten /  
Consult instructions for use

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**I. FOREWORD**

The instructions for use are valid for the **LARYVOX® SOS mask**. They serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

**II. INTENDED USE**

The ventilation funnel **LARYVOX® SOS mask** enables safe mouth-to-tracheostoma ventilation, whenever required or in emergencies.

**III. WARNING**

The **LARYVOX® SOS mask** is a single-patient product and is thus intended exclusively for one-time or repeated use in one individual patient.

It is recommendable to carry the ventilation funnel **LARYVOX® SOS mask** at all times, in order to have it close to hand in emergencies.

**IV. PRODUCT DESCRIPTION**

The ventilation funnel consists of a soft silicone tracheostoma mask and a plastic adapter tube, which is attached to the mask. An emergency pass and an emergency sticker for neck breathers are included in the scope of delivery.

**V. INSTRUCTIONS****Performing ventilation (instructions for the resuscitator):**

1. Attach the plastic adapter tube to the tracheostoma mask ❶.
2. Place the ventilation funnel **LARYVOX® SOS mask** directly on the neck opening or on the tracheostomy tube inside it.
3. Hold the ventilation funnel firmly on the plastic tube and press the mask on the tracheostoma such that a complete seal between the skin and silicone tracheostoma mask is achieved ❷.
4. Blow air into the plastic tube; in case of successful resuscitation the chest rises ❸.
5. Release the plastic tube for expiration. The expired air can now escape, the chest falls ❹.
6. Continue resuscitation (12 x per minute).
7. Ensure that a physician is notified.

**VI. CLEANING**

**LARYVOX® SOS mask** has to be thoroughly cleaned with clear water after each use. Do not put in a dishwasher or disinfect with boiling water!

**VII. SERVICE LIFE**

Do not use the **LARYVOX® SOS mask** if it is damaged.

**VIII. LEGAL NOTICES**

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages (especially not for functional deficiencies, injuries, infections, and/or other complications or adverse events) caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use.

This applies – to the extent permitted by law – both to damages to the products themselves caused thereby and to any consequential damages caused thereby.

Product specifications described herein are subject to change without notice.

**LARYVOX®** is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.