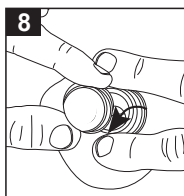
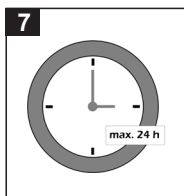
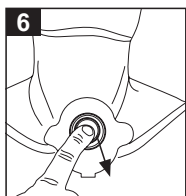
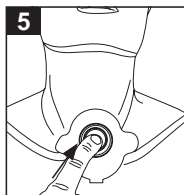
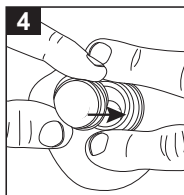
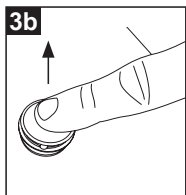
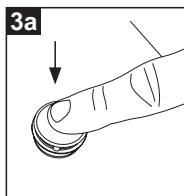
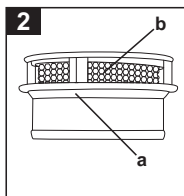
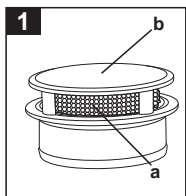
LARYVOX® EXTRA HME

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EN

I. FOREWORD

These instructions for use are valid for LARYVOX® EXTRA HME filter cassettes.

The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

II. INTENDED USE

LARYVOX® EXTRA HME filter cassettes are particle filters consisting of a foam filter and a plastic housing.

The LARYVOX® EXTRA HME filter cassettes are affixed over the tracheostoma either on a special base plate with holding ring or on a tracheostomy tube with 22-mm combi-adaptor.

LARYVOX® EXTRA HME filter cassettes filter the breathing air, thereby reducing intrusion of particles into the patient's airways.

The filter cassettes reduce the formation of viscous secretions in the lungs. They store the moisture and warmth of the exhaled air in the filter medium and emit these to the inhaled air again during inspiration.

When used for the first time the products must only be selected, used and inserted by clinicians or medical professionals trained in their use.

III. WARNINGS

LARYVOX® EXTRA HME are single-patient products and are intended for single use only.

CAUTION!

Cleaning, disinfection, re-sterilisation and re-use can impair the safety and function of the product and are therefore not permissible!

The speaking button may only be used exclusively by patients with a partially or fully preserved larynx as well as persons wearing voice prostheses (shunt valve)!

The LARYVOX® EXTRA HME filter cassette must under no circumstances be inserted directly into the tracheostoma!

IV. PRODUCT DESCRIPTION

1. LARYVOX® EXTRA HME filter cassette (REF 49860)

- Normal respiratory resistance
- For patients directly after laryngectomy
- For situations with normal physical exertion

2. LARYVOX® EXTRA HME MEDIUM filter cassette (REF 49862)

- Average respiratory resistance
- Physical activities

3. LARYVOX® EXTRA HME HighFlow filter cassette (REF 49861)

- Low respiratory resistance
- For new users of HMEs
- For situations with strong physical exertion

4. LARYVOX® EXTRA HME SPORT filter cassette (REF 49863)

- Very low respiratory resistance
- For sports activities such as, for example, endurance sports

LARYVOX® EXTRA HME filter cassettes feature:

- Plastic housing with an outer diameter of 22 mm
- Compatible with all commonly used 22 mm receptacle systems
- A foam filter for heat and moisture exchange (1a)
- A speaking button (1b), by means of which the tracheostoma can easily be closed for speaking
- A finger groove (2a) to enable easy removal of the HME filter cassette from the tracheostoma housing for coughing
- Side openings for the airflow to enable unhindered and easy breathing (2b)

V. COMPLICATIONS

If the particle filter is clogged or if signs of clogging appear, the LARYVOX® EXTRA HME filter cassette must immediately be removed and disposed of.

VI. PRECAUTIONS

Care must be taken to ensure that the valve opening is not blocked by clothing or the like in order to enable unhindered breathing (risk of respiratory distress).

VII. CONTRAINDICATIONS

The LARYVOX® EXTRA HME should be removed if respiratory resistance is too high.

Use in patients with strong exsiccosis (dehydration), or excessive secretion in the lungs and airways, or strong air leakage flow (exhaled air not flowing through the filter cassette) is not permissible.

The HME is intended for use on patients with spontaneous breathing.

For persons with impaired consciousness and/or persons requiring a high level of care, monitoring by trained specialist personnel is required to detect excessive breathing resistance in time and to initiate the appropriate measures.

VIII. INSTRUCTIONS

- Check whether the packaging and the product are in a faultless condition.
- Check the use-by or expiry date. Do not use after this date.
- Press the speaking button down in order to check whether it is fully functional (Ⓐa). When the speaking button is released, it immediately returns to the open starting position (Ⓐb).
- Fit Ⓐ the LARYVOX® EXTRA HME filter cassettes vertically onto the 22-mm connection of the tracheostomy tube or base plate, exerting gentle pressure.
- Check the respiratory resistance.
- For speaking, press the speaking button Ⓐ down slightly with your finger so as to enable phonation by means of the closure.
- When the speaking button is released, it immediately returns to its open breathing position (starting position) Ⓐ.
- Remove the LARYVOX® EXTRA HME filter cassettes after a Ⓐ wearing period of 24 hours at most, or if the filter cassette is contaminated with secretions for instance. To do this, hold the base plate or the tracheostomy tube in position with two fingers Ⓐ and remove the filter cassette by tilting it downward slightly.

IX. SERVICE LIFE

The maximum period of use/wearing period is 24 hours.

The filter must be changed even several times a day if required, for instance if the respiratory resistance is in danger of increasing due to accumulation of secretions.

X. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH accepts no liability for damage caused by unauthorised product modifications, repairs not carried out by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products contrary to the provisions of these instructions for use.

Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

Product specifications described herein are subject to change without notice.

LARYVOX® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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