**LARYNGOFIX® / TRACHEOFIX® /
TRACHEOSTOMA-FIX® / LARYTAPE®**

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PICTROGRAM LEGEND

en

Pictograms listed below you'll find on the product packaging, if applicable.

	Catalogue number		Keep dry
	Batch code		Do not reuse
	Use by		Medical devices
	Date of manufacture		Non-sterile
	Content (in pieces)		CE marking
	Consult instructions for use		Manufacturer
	Keep away from sunlight		

TRACHEOSTOMA PROTECTION

FOREWORD

These instructions for use are valid for **LARYNGOFIX® / TRACHEOFIX® / TRACHEOSTOMA-FIX® / LARYTAPE®** tracheostoma protection.

These instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

Please read the instructions for use carefully before using the product for the first time!

Keep them in an easily accessible place for future reference.

Please keep the package for as long as you use the product.

It contains important information on the product!

1. INTENDED PURPOSE

LARYNGOFIX® / TRACHEOFIX® / TRACHEOSTOMA-FIX® / LARYTAPE® filter the breathing air, thereby reducing intrusion of particles into the patient's airways.

They store the moisture and warmth of the exhaled air in the filter medium and emit these to the inhaled air again during inspiration.

LARYNGOFIX® TOUCH / LARYTAPE® TOUCH enables closing the tracheostoma with a button and thus phonation.

2. SAFETY INFORMATION

LARYNGOFIX® / TRACHEOFIX® / TRACHEOSTOMA-FIX® / LARYTAPE® are single-patient products and are intended for single use only.

It must not be cleaned or disinfected, as use of cleaning agent can render the filter medium unusable.

Sterilisation and reuse can impair the function and is therefore not permissible either.

3. INDICATION

The products are indicated for tracheotomized and laryngectomized patients.

4. CONTRAINDICATION

Do not use if the patient is allergic to the material.

5. COMPLICATIONS AND ADVERSE REACTIONS

Should you experience skin Irritation (e.g. redness, itchiness), discontinue the product immediately and consult a physician.

This may be due to special skin sensitivity or allergy-related incompatibility.

6. PRODUCT DESCRIPTION

The tracheostoma protector is a functional filter for tracheostomised and laryngectomised patients and ensures that the breathing air is filtered, humidified and warmed.

The products also provide good coverage and reliable protection against the ingress of dirt particles.

We supply **LARYNGOFIX®** / **TRACHEOFIX** / **TRACHEOSTOMA-FIX®** / **LARYTAPE®** in different sizes.

7. APPLICATION

Clean the skin around the tracheostoma in front of a mirror using warm water and pH-neutral soap or **OPTIFAHL®** cleaning cloths.

Dry the skin.

Before cleaning, remove residues of adhesive from the skin if necessary, using **OPTICLEAR®**.

After removing the protective foil backing (1) place the tracheostoma protection centred over the tracheostoma (2) and cover the tracheostoma this way (3).

Remove the tracheostoma protection (4) by carefully removing the adhesive strip from the skin.

LARYNGOFIX® TOUCH / **LARYTAPE® TOUCH**:

For speaking, press the button down slightly with your finger so as to enable phonation by means of the closure (5)

CAUTION!

Do not use any fatty creams!

Otherwise, secure attachment is no longer ensured.

8. HYGIENE INSTRUCTIONS

LARYNGOFIX® / **TRACHEOFIX®** / **TRACHEOSTOMA-FIX®** / **LARYTAPE®** are single-patient products and are intended for single use only.

It must not be cleaned or disinfected, as use of cleaning agent can render the filter medium unusable.

Sterilisation and reuse can impair the function and is therefore not permissible either.

9. STORAGE

This product should be stored in a dry environment away from sunlight and/or heat.

10. SERVICE LIFE

The maximum period of wearing period is 24 hours.

The tracheostoma protection can also be changed several times a day, as necessary.

Deviations from the expected wearing period occur as a result of individual wearing conditions and skin properties.

11. DISPOSAL

The product must only be disposed of in accordance with the national regulations.

12. LEGAL NOTICES

en

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for functional deficiencies, injuries, infections, and/or other complications or adverse events caused by unauthorised product alterations or improper use, care, and/or handling.

In particular, Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by modifications to the product or by repairs, if these modifications or repairs were not carried out by the manufacturer himself.

This applies – to the extent permitted by law – both to damages to the products themselves caused thereby as well as to any consequential damages caused thereby.

When using the product beyond the period of use specified under Section 10, and/or use, treatment, maintenance (cleaning, disinfection) or storage of the product in non-compliance with the specifications laid down in these instructions for use, Andreas Fahl Medizintechnik-Vertrieb GmbH will, as far as legally permissible, be free of any liability, including liability for defects.

Should a serious incident occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, then this must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

Sale and delivery of all Andreas Fahl Medizintechnik-Vertrieb GmbH products is carried out exclusively in accordance with our General Terms and Conditions of Business (GTC) which can be obtained directly from Andreas Fahl Medizintechnik-Vertrieb GmbH.

The manufacturer reserves the right to make changes to the product at any time without notice.

LARYNGOFIX® / TRACHEOFIX® / TRACHEOSTOMA-FIX® / LARYTAPE® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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