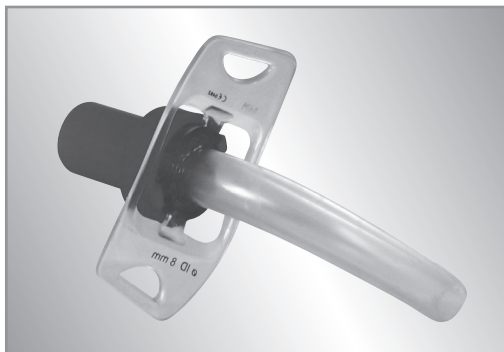
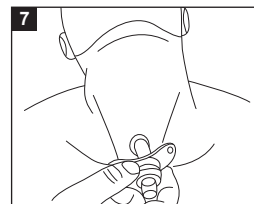
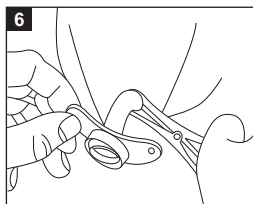
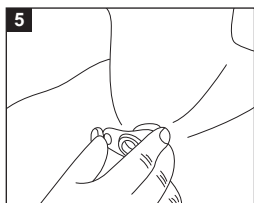
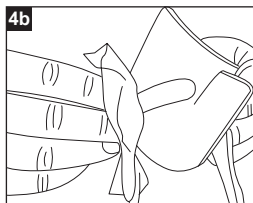
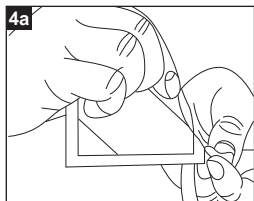
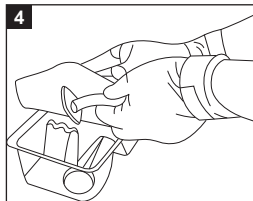
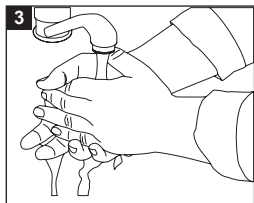
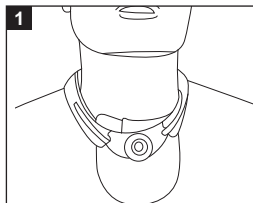
FAHL

FAHL® BIESALSKI

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PICTOGRAM LEGEND

Pictogrammes listed below you'll find on the product packaging, if applicable.

With two inner cannulas

Content (in pieces)

Outer cannula including 15 mm-connector (UNI)

Single patient use

Inner cannula with low profile

Do not use if package is damaged

Speaking valve (PHON)

Keep dry

Tube holder included

Keep away from sunlight

Specially for children

Medical devices

MRT suitable

Manufacturer

Date of manufacture

Use by

Consult instructions for use

Batch code

Catalogue number

Sterilized using ethylene oxid

Do not resterilize

FAHL® BIESALSKI TRACHEOSTOMY TUBE

EN

I. FOREWORD

These instructions for use are valid for Fahl® tracheostomy tubes. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling of Fahl® tracheostomy tubes.

Please read the instructions for use carefully before using the product for the first time!

Keep them in an easily accessible place for future reference.

Please keep the package for as long as you use the tracheostomy tube. It contains important information on the product!

II. INTENDED USE

Fahl® tracheostomy tubes are intended to stabilise the tracheostoma following laryngectomy or tracheostomy.

The tracheostomy tube is designed to keep the tracheostoma open.

When used for the first time the products must only be selected, used and inserted by clinicians or medical professionals trained in their use.

III. WARNINGS

Patients must be trained by medical professionals on how to use the Fahl® tracheostomy tubes safely.

Fahl® tracheostomy tubes must never be blocked, e.g. by mucus or encrustation. Blockage can cause asphyxiation!

Mucus in the trachea can be aspirated (removed by suction) through the tracheostomy tube with a tracheal suction catheter.

Damaged tracheostomy tubes must not be used and must be discarded immediately. Use of a damaged tube may result in airway compromise.

When the tracheostomy tubes are inserted or removed, this can lead to irritations, coughing or slight bleeding. If bleeding persists, consult your doctor immediately!

Tracheostomy tubes that contain metal parts must under no circumstances be used during a radiation therapy (radiotherapy), as this can lead to serious skin damage! If you need to wear a tracheostomy tube during a radiation therapy, then only use plastic tracheostomy tubes without any metal parts. In the case of plastic tracheostomy tubes with silver speaking valve, the valve can for instance be removed completely from the tube, including the safety chain, by removing the inner cannula from the outer cannula before starting a radiation therapy.

Unintentional Inhalation of a tube that was not sized correctly may need to be removed by a doctor/physician.

CAUTION!

Tracheostomy tubes with speech function are only recommended for tracheotomised patients with normal secretion and normal mucous tissue.

IV. COMPLICATIONS

The following complications could emerge when using this device:

Contamination of the stoma may make it necessary to remove the tube. Contaminations can furthermore cause infections that necessitate the use of antibiotics.

If mucus blocks the tube it should be removed and cleaned.

V. CONTRAINDICATIONS

Do not use if the patient is allergic to the material.

CAUTION!

Tracheostomy tubes with speaking valves must not be used on any account by laryngectomised patients, as this could lead to serious complications, even suffocation!

VI. PRECAUTIONS

The correct tracheostomy tube size should be selected by the treating physician or by trained medical professionals.

We highly recommend always keeping at least two replacement tubes on hand to ensure continuous supply.

VII. PRODUCT DESCRIPTION

The Fahl® Biesalski tracheostomy tubes are products that are made of a variety of medical-grade synthetic materials.

The tracheostomy tubes are made of thermosensitive medical synthetics that develop their optimal product properties at body temperature.

We supply the Fahl® tracheostomy tubes in various sizes and lengths.

Please see the size table in the appendix.

Fahl® tracheostomy tubes are reusable medical products intended for single-patient use.

Fahl® tracheostomy tubes are only to be used by the same patient and not by another patient. The package contains 1 tube, supplied sterile by ethylene oxide sterilization (EO). It may be advisable to use tubes of different lengths alternately so the tip of the tube does not always touch the same spot in the trachea and cause irritation. This prevents pressure points or the formation of granulation tissue. Always discuss the exact procedure with your doctor/physician.

MRT (Magnetic Resonance Tomography) INFORMATION

These tracheostomy tubes can even be worn by the patient during magnetic resonance tomography (MRT).

1. Neck flange

The size specifications are printed on the neck flange.

The neck flange of the tracheostomy tubes has two side eyelets for inserting a tube holder.

All Fahl® tracheostomy tubes with fastening eyelets include a disposable tube holder. The tube holder secures the tracheostomy tube to the neck.

Please follow the instructions provided with the tube holder carefully when attaching and removing it from your tracheostomy tube.

Ensure that the Fahl® tracheostomy tube is placed in the tracheostoma without tension and the position is not changed when fastening the tube holder.

2. Connectors and adapters

Connectors and adapters are used to connect compatible tube accessories.

The options available to a patient depend on the clinical condition, such as status post laryngectomy or tracheostomy.

The inner cannula is fixed in place by means of bayonet locks, which are located on the connector, the speaking valve and the cough cap respectively. The connector in this case is the universal attachment (15 mm connector), with the aid of which so-called artificial noses or heat and moisture exchangers (HMEs, filters for exchanging heat and moisture) can be attached.

3. Cannula tube

The cannula tube borders directly on the neck flange and directs the airflow into the trachea.

The tube tip is rounded to prevent irritation of the mucus membranes in the trachea.

The swivel connector serves to securely attach compatible accessories: the accessories are fixed in place with only a slight rotating movement in clockwise direction (in inserted condition from the patient's point of view), and thus without having to exert much tension on the tracheostomy tube.

4. Inner cannula

The inner cannulas can easily be removed from the outer cannula, thus allowing the air supply to be increased quickly if necessary (for instance in case of dyspnoea).

Inner cannulas must never be used without outer cannula but must always be affixed to the outer cannula.

4.1 Speaking valves

Tracheostomy tubes with speaking valve (LINGO/PHON) are used after tracheotomy with a complete or partially retained larynx and enable the user to speak.

VIII. DIRECTIONS FOR TUBE INSERTION AND REMOVAL

For the Doctor/Physician

The appropriate tracheostomy tube must be selected by a doctor/physician or trained medical professionals.

Select a tube that fits the patient's anatomy to optimise comfort and ventilation (breathing in and out).

The inner cannula can be removed at any time to increase air supply or for cleaning. This can for instance be necessary if the cannula is clogged up with secretion residues which cannot be removed by coughing or because no equipment for suctioning off the secretions is available.

For the Patient

CAUTION!

Carefully examine the sterile packaging to ensure that it has not been tampered with or damaged. Do not use the product if the packaging has been damaged.

Check the use-by or expiry date. Do not use after this date.

It is advisable to use sterile disposable gloves.

Carefully examine the tube before first use to make sure that it is not damaged and that there are no loose parts.

Should you notice any anomaly or anything unusual, DO NOT use the tube. Return the tube to the manufacturer for inspection.

The tube must always be cleaned and, if necessary, disinfected as follows before re-inserting according to the instructions provided below.

If secretion collects in the lumen of the Fahl® tracheostomy tube or stoma button and cannot be removed by coughing or aspiration, the tube should be removed and cleaned.

After cleaning and/or disinfection, carefully examine the Fahl® tracheostomy tube for sharp edges, cracks, or other signs of damage, since these may impair function and/or injure the mucus membranes in the airways.

Never under any circumstances continue using damaged tracheostomy tubes.

WARNING!

Patients must be briefed by trained medical professionals in the safe use of the Fahl® tracheostomy tubes.

1. Insertion of the tube

Step-by-step instructions to insert Fahl® tracheostomy tubes.

Before application, users should clean their hands (see picture 3).

Remove tube from the package (see picture 4).

If you are inserting the tube yourself, use a mirror to make insertion of the Fahl® tracheostomy tube easier.

When inserting the Fahl® tracheostomy tube, hold it by the neck flange with one hand (see picture 5).

Pull the tracheostoma slightly apart with your free hand to allow the tip of the tube to fit into the tracheostoma more easily.

Special aids for dilating the tracheostoma are also available. These allow the tracheostoma to be dilated gently and evenly, for instance also in emergency situations with collapsing tracheostoma (see picture 6).

Make sure that the tube is not damaged by friction when using an instrument for assistance.

Now carefully insert the tube into the tracheostoma during the inspiration phase (while breathing in) while tilting your head slightly back (see picture 7).

Advance the tube into the trachea.

Straighten your head once the tube has been inserted further into the trachea.

The tracheostomy tubes should always be attached with a special tube holder. This stabilises the tube and thus ensures that the tracheostomy tube is securely seated in the tracheostoma (see picture 1).

Then a tracheal compress is pushed onto the tracheostomy tube.

To increase the lubricity of the tracheostomy tube and thus facilitate insertion into the trachea, it is recommended to wipe the outer tube with an OPTIFLUID® stoma oil wipe (REF 31550), which allows even distribution of the stoma oil on the tube (see Figs. 4a and 4b) or FAHL® OPTIFLUID® Lubricant Gel 20g tube (REF 36100) or FAHL® OPTIFLUID® Lubricant Gel 3g sachet (Ref 36105).

2. Removing the tube

CAUTION!

Accessories such as the speaking valve or connectors/cough cap must be removed first before proceeding to remove the Fahl® tracheostomy tube.

CAUTION!

If the tracheostoma is unstable, or in emergency situations (puncture/dilation tracheostomy), the tracheostoma can collapse after withdrawal of the tracheostomy tube, thereby impairing air supply. A fresh tracheostomy tube must be kept ready for use in such cases and must be quickly inserted if necessary. A tracheal dilator (REF 35500) can be used for temporarily securing the air supply.

Proceed very carefully to avoid injury to the mucus membranes.

Step-by-step instructions to remove the Fahl® tracheostomy tubes:

The tracheostomy tubes should be removed with the head slightly tilted back. Grip the tube at the side by the neck flange or the housing (see picture 7).

Carefully remove the tracheostomy tubes.

When removing the inner cannula, the following must be observed:

The accessories (adapter/speaking valve/cough cap) must first be removed by slight counterclockwise rotation (in inserted condition from the patient's point of view).

The inner cannula can now be removed from the outer cannula by pulling it gently.

Reinsertion of the inner cannula is performed in reverse sequence as described above.

IX. CLEANING AND DISINFECTION

CAUTION!

For reasons of hygiene and to avoid the risk of infection, you should thoroughly clean the Fahl® tracheostomy tube at least twice a day, more often during heavier secretion production.

CAUTION!

The tracheostomy tubes must not be cleaned using a dishwasher, a steam cooker, a microwave oven, a washing machine or similar appliances!

Keep this in mind, your personal cleaning schedule, including additional disinfection if required, must always be determined in consultation with your doctor/physician and according to your individual needs.

Regular disinfection is only required if this is indicated from the medical point of view as ordered by the doctor/physician. The reason for this is that the upper airways are not free of microbes in healthy patients either.

General cleaning may be insufficient to meet the special hygiene requirements to prevent infections for patients with specific problems (e.g. MRSA, ORSA etc.), who are subject to an increased danger of re-infection. We recommend chemical disinfection of the tubes according to the instructions provided below. Please consult your doctor/physician.

CAUTION!

Cleaning agent and disinfectant residues on the tracheostomy tube can lead to irritations of the mucus membranes or other health impairments.

Tracheostomy tubes are considered as belonging to the category of instruments with hollow cavities. Special care must therefore be taken when performing cleaning or disinfection that the tube is fully wetted with the solution used and is unobstructed (see picture 8).

The solutions used for cleaning and disinfection must be freshly prepared prior to each use.

1. Cleaning

Fahl® tracheostomy tubes must be cleaned/replaced regularly in accordance with the individual patient's needs.

Only use cleaning agents if the tube is outside the tracheostoma.

A mild, pH-neutral washing lotion can be used for cleaning the tube. We recommend using the special cannula-cleaning powder (REF 31110) as directed by the manufacturer.

Never clean Fahl® tracheostomy tubes with cleaning agents that are not approved by the manufacturer. Never use aggressive household cleaners, detergents, high-concentration alcohol or denture cleaners.

This may be very hazardous to health. The tube may also be destroyed or damaged.

Tubes can also be cleaned by thermal disinfection at not more than 65 °C. Use clean water with a temperature of not more than 65 °C for this purpose. Make sure that the temperature is kept constant (use a thermometer to monitor the temperature) and never under any circumstances expose the tubes to boiling water for disinfection. This could seriously damage the tracheostomy tube.

Cleaning Steps

Remove any additional items before cleaning.

The inner cannula must also be removed from the outer cannula.

First, rinse the tube thoroughly under running water (see picture 9).

Use only lukewarm water to prepare the cleaning solution and observe the instructions for use of the cleaning agent.

We recommend the use of the cannula-cleaning tub with sieve insert (REF 31200) to facilitate the cleaning procedure.

To do this, hold the sieve insert by the upper edge to prevent contact with and contamination of the cleaning solution (see picture 10).

Place only one tracheostomy tube at a time in the sieve insert of the cannula-cleaning tub. If more than one tube is cleaned at a time, they may be compressed too strongly and damaged by excessive pressure.

You can place the inner and outer cannula side by side for this procedure.

Immerse the sieve insert with the tracheostomy tube components in the prepared cleaning solution.

After the immersion time has expired (see instructions for use of the cannula cleaning powder), wash the tracheostomy tube thoroughly several times with lukewarm, clear water (see picture 9). There must be no residues of the cleaning agent on the tube when it is inserted into the tracheostoma.

If necessary, for example, if persistent secretion residues cannot be removed by the cleaning bath, additional cleaning with a special cannula-cleaning brush (OPTIBRUSH®, REF 31850 or OPTIBRUSH® Plus with fibre top, REF 31855) may help. Only use the cleaning brush, if the tube is removed and already outside the tracheostoma.

Always insert the cannula cleaning brush into the cannula from the cannula tip (see picture 11).

Use the brush as directed and proceed with great care to avoid damage to the soft tube material.

In tracheostomy tubes with speaking valve, the speaking valve must first be detached from the inner cannula. The valve itself must not be cleaned with the brush since it could otherwise be damaged or break off.

Thoroughly rinse the tube or button under lukewarm running water or sterile saline solution (0.9% NaCl solution).

Dry the tube thoroughly with a clean and lint-free cloth after wet cleaning.

Never use tracheostomy tubes with impaired functionality or with damages such as sharp edges or cracks, because this may lead to injuries of the mucus membranes in the trachea. If damages are found, the tube must not be used under any circumstances.

The accessories (adapter/speaking valve/cough cap/decannulation stopper) can be cleaned in the same way as the tracheostomy tube.

2. Chemical Disinfection Instructions

The Fahl® tracheostomy tube can be disinfected by cold disinfection with special chemical disinfectants.

Disinfection should always be done if ordered by a doctor/physician due to specific health concerns caused by disease, infection, or your specific situation.

Disinfecting is generally indicated to prevent cross-infection and in inpatient situations (e.g. hospitals, nursing homes, and/or other health care facilities) to limit infections.

CAUTION

Always clean according to the cleaning procedure described above before proceeding to disinfection (if applicable).

Disinfectants that release chlorine or that contain strong alkalis or phenol derivatives must under no circumstances be used. This could badly damage or even destroy the tracheostomy tube.

Disinfection steps

For this purpose, OPTICIT® tube disinfectant (REF 31180) should be used in accordance with the manufacturer's instructions.

As alternative, we recommend a disinfectant based on glutaric aldehyde as active ingredient. Always observe the area of application and spectrum of activity specified for the disinfectant by the manufacturer.

Follow the instructions for the disinfectant.

Dry the tube thoroughly with a clean and lint-free cloth after wet cleaning.

3. Sterilisation/Autoclaving

Resterilisation is not allowed.

CAUTION!

Heating to over 65°C, disinfection with boiling water or steam sterilisation are not allowed and lead to damage of the tracheostomy tube.

X. STORAGE/CARE

Currently not used and cleaned tubes should be stored in a clean plastic container, dry environment away from dust, sunlight and heat.

Still sterile packed replacement tubes should be stored in a dry environment away from dust, sunlight and heat.

After the inner cannula has been cleaned, disinfected if necessary and then dried, the outer surface of the inner cannula should be lubricated by applying stoma oil (OPTIFLUID® Stoma Oil, 25 ml bottle REF 31525/Stoma oil wipe REF 31550) or lubricant gel (FAHL® OPTIFLUID® Lubricant Gel 20 g tube REF 36100/3 g sachet REF 36105).

We highly recommend keeping at least two replacement tubes on hand to ensure continuous supply.

XI. SERVICE LIFE

These tracheostomy tubes are sterile products for single-patient use.

The maximum period of use is 29 days (reckoned from the date on which the sterile package was opened). If a pathogenic microbe (e.g. MRSA) is present, the period of use/indwelling time is reduced accordingly.

The service life of a tube is influenced by many factors. The composition of the secretions, the quality of cleaning and other aspects, for example, are very important.

The maximum service life is not extended by using the tube intermittently (e.g. alternately with other tubes at normal cleaning intervals).

CAUTION!

Modifications to the tracheostomy tube of any kind, in particular perforations as well as repairs to the tube, may only be carried out by the manufacturer himself or by companies expressly authorised for this in writing by the manufacturer! Inexpertly performed work on tracheostomy tubes can lead to serious injuries.

XII. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for functional deficiencies, injuries, infections, and/or other complications or adverse events caused by unauthorised product alterations or improper use, care, and/or handling.

In particular, Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by modifications to the tracheostomy tube, especially due to shortening of the same or perforations, if these modifications or repairs were not carried out by the manufacturer himself. This applies both to damages to the tubes themselves caused thereby and to any consequential damages caused thereby.

If the tracheostomy tube is used beyond the period of use specified under Section XI, and/or if the tracheostomy tube is used, maintained (cleaned, disinfected) or stored in non-compliance with the instructions and specifications laid down in these instructions for use, Andreas Fahl Medizintechnik-Vertrieb GmbH will, as far as legally permissible, be free of any liability, including liability for defects.

Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

Sale and delivery of all Andreas Fahl Medizintechnik-Vertrieb GmbH products is carried out exclusively in accordance with our General Terms and Conditions of Business which can be obtained directly from Andreas Fahl Medizintechnik-Vertrieb GmbH.

Product specifications described herein are subject to change without notice.

FAHL® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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All measurements listed in the tables below are for standard cannulas.
Customised cannulas are available upon request.

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Sizing information:

All lengths in mm | GR = Size | AD1 = Outer diameter at the neck flange | AD2 = Outer diameter at cannula tip | ID1 = Inner diameter at the cannula tip of the outer cannula | ID2 = Inner diameter at the cannula tip of the inner cannula | L1 AB = Length over the outer curve | L2 MB = Length over the middle curve | Ø-CD = CUFF diameter | Ø-BW = Bending angle

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