TRACHEOTEX®

SCARF / EASY / BIB PED / ROLLI / SHIRT / BIB

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REF

- 40100 TRACHEOTEX® BIB 3L 34 - 46 cm
 40110 TRACHEOTEX® BIB 3L PED 24 - 33 cm
 40200 TRACHEOTEX® BIB 5L 34 - 46 cm
 40210 TRACHEOTEX® BIB 5L PED 24 - 33 cm
 40300 TRACHEOTEX® BIB 8L 34 - 46 cm
 40400 TRACHEOTEX® BIB 12L 34 - 46 cm
 41700 TRACHEOTEX® Protection BIB large 21 x 19 cm
 41800 TRACHEOTEX® Protection BIB small 15 x 12 cm
 42000 TRACHEOTEX® Scarf 41 - 47 cm
 42002 TRACHEOTEX® EASY max. 55 cm
 42010 TRACHEOTEX® Scarf PED
 43000 TRACHEOTEX® Rolli Klett Fine
 43001 TRACHEOTEX® Rolli Klett Jersey
 43300 TRACHEOTEX® Rolli min. 18 cm
 43301 TRACHEOTEX® Shirt min. 20 cm
 43302 TRACHEOTEX® Rolli Zip Fine

PIKTOGRAMM-LEGENDE

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
PICTROGRAM LEGEND

Pictogrammes listed below you'll find on the product packaging, if applicable.


	Catalogue number		For reuse on a single patient
	Batch code		Medical devices
	Date of manufacture		CE marking
	Content (in pieces)		Manufacturer
	Consult instructions for use		Do not iron
	Keep away from sunlight		Hand wash
	Keep dry		Do not dry in the drum dryer

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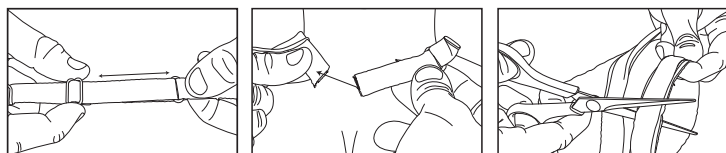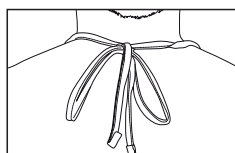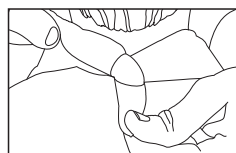
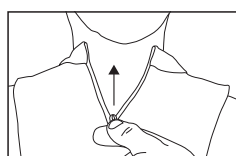
TRACHEOSTOMA PROTECTION

1. FOREWORD

These instructions are valid for **TRACHEOTEX® Scarf / EASY / BIB PED / ROLLI / SHIRT / BIB**.

The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

Please read the instructions for use carefully before using the product!

Please keep the package for as long as you use the product.

2. SAFETY INFORMATION

TRACHEOTEX® are single-patient products and are thus intended exclusively for one-time or repeated use in one individual patient.

3. INTENDED PURPOSE

TRACHEOTEX® are designed to be used for covering the tracheostoma and prevent the intrusion of foreign objects into the airways.

At the same time, the breathing air is filtered, warmed and humidified, and escaping tracheal secretions are absorbed.

4. INDICATION

The product is indicated for tracheotomized and laryngectomized patients.

5. CONTRAINDICATION

Do not use if the patient is allergic to the material used.

6. COMPLICATIONS AND ADVERSE REACTIONS

If the **TRACHEOTEX®** is damaged or show signs of wear (e.g. fraying or the like), it must be replaced!

If the product is damaged, its function can be impaired!

The **TRACHEOTEX®** must not be worn too tightly around the neck. Approx. 1 cm clearance is ideal.

7. PRODUCT DESCRIPTION AND APPLICATION

TRACHEOTEX® are placed around the neck and completely cover the tracheostoma.

They must not be worn too tightly around the neck, approx 1 cm clearance is ideal.

The incorporated inserts are positioned over the tracheostoma during application and cover the stoma completely (this does not apply to **TRACHEOTEX® ROLLI** and **TRACHEOTEX® Shirt**).

The **TRACHEOTEX®** is a tracheostoma protection with foam for neck breathers.

The following products are available for covering the tracheostoma: (please refer to Figures for diagrams of the different applications).

The **TRACHEOTEX® Scarf** protective stoma neckerchief has an insert made of a fine gauze fabric which warms up and filters the inhaled air.

An adjustment clip on the fixing strap allows the neckerchief to be adjusted to different neck widths.

The neckerchief is fastened by means of a hook-and-loop fastener, the hook-and-loop fastener can be shortened with the aid of scissors to an appropriate length to fit the user.

The **TRACHEOTEX® EASY** stoma protection neckerchief is designed in the shape of a triangular bandage.

The neckerchief is fastened by knotting together the tips at the back of the neck.

The **TRACHEOTEX® PROTECTION BIB** is a tracheostoma protection with foam for neck breathers.

Place the fabric straps of the **TRACHEOTEX® PROTECTION BIB** around your neck.

Now knot the fabric straps with a bow.

The **TRACHEOTEX® BIB PED** stoma protection neckerchief for children is supplied in a small size with shortened fastening straps.

The neckerchief is fastened by means of a hook-and-loop fastener.

The hook-and-loop fastener can be shortened with the aid of scissors to an appropriate length to fit the user.

The **TRACHEOTEX® ROLLI**s / Shirts are slipped over the head and are worn underneath a shirt or a jacket.

Thanks to the elastic material, they adapt to different neck widths.

The **TRACHEOTEX® ROLLI**s **ZIP** feature a zip fastener on the back, and are available in two different materials: a coarsely-textured and a finely-textured fabric.

TRACHEOTEX® ROLLIs **ZIP** are pulled down over the head and the zip fastener is closed.

The **TRACHEOTEX® BIB** stoma protection **BIB**, which consists of multilayer fine gauze fabric, is available in different thicknesses: 3-layer, 5-layer, 8-layer, 12-layer, 16-layer.

The thinner the **BIB**, the greater the air permeability.

An adjustment clip on the fixing strap allows the neckerchief to be adjusted to different neck widths, the neckerchief is fastened by means of a hook-and-loop fastener.

8. HYGIENE INSTRUCTIONS

TRACHEOTEX® are washable up to 30°C; do not wring, tumble-dry or bleach.

We recommend hand wash.

Smooth the **TRACHEOTEX®** and allow to dry between two towels.

This avoids creasing of the foam layer.

NOTE!

The **TRACHEOTEX®** should not be washed more than 3 times.

TRACHEOTEX® SCARF, ROLLI, SHIRT, EASY

Only iron the **TRACHEOTEX®** at low temperature.

TRACHEOTEX® BIB PED, Protection BIB

Do not iron!

It is not permissible to machine-wash, tumble-dry or bleach the **TRACHEOTEX®**.

More frequent washing could cause the foam to lose strength and flocculate.

9. STORAGE

TRACHEOTEX® should be stored in a dry environment away from sunlight and/or heat.

10. SERVICE LIFE

TRACHEOTEX® must be changed every day for reasons of hygiene.

They are intended for repeated use and can be cleaned (washed).

The service life depends on the individual frequency of use and on individual handling.

The products must be replaced immediately when damaged.

It is intended for repeated use and should not be washed more than 3 times.

The product has to be replaced immediately in case of any damage.

11. DISPOSAL

The product must only be disposed of in accordance with the national regulations.

12. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH accepts no liability for damage caused by unauthorised product modifications, repairs not carried out by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products contrary to the provisions of these instructions for use.

This applies – to the extent permitted by law – both to damages to the products themselves caused thereby as well as to any consequential damages caused thereby.

The manufacturer reserves the right to make changes to the product at any time without notice.

TRACHEOTEX® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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